



ENZYMAX EARTH DUAL ENZYME ULTRASONIC DETERGENT A SUPERIOR LEVEL OF INSTRUMENT CLEANING

Dual Enzyme Ultrasonic Detergent is specifically formulated for efficient ultrasonic cleaning or pre-soaking of cassettes and instruments. Enzymax Earth Dual Enzyme Ultrasonic Detergent offers the same superior cleaning attributes as Enzymax with an increased awareness of our environment. All Enzymax Earth products are biodegradable and phosphate free. Can also be used as an evacuation system cleaner. Contains Steelgard agents to protect instruments from harmful minerals naturally found in water that contribute to buildup, spotting and corrosion.

ECO-FRIENDLY FORMULA*



ENZYMAX EARTH LIQUID

Gallon Bottle	IMS-1336
Makes 128 Gallons/485 liters	

ENZYMAX EARTH PAX SINGLE USE PACKETS

32 Dissolvable Single-Use Powder Packets	IMS-1332
Makes 32 Gallons/121 liters	

96 Dissolvable Single-Use Powder Packets	IMS-1333
Makes 96 Gallons/363 liters	

*Biodegradable formula according to Organisation for Economic Cooperation & Development (OECD) Testing

HU-FRIEDY SOCIAL RESPONSIBILITY

At Hu-Friedy, we believe in helping dental professionals perform at their best. We strongly believe in protecting the health of our planet in addition to supporting the health of individuals. Part of this effort is looking toward the future and finding solutions such as printing this flyer on recycled paper.

Hu-Friedy is proud to join a number of proactive, forward-thinking companies who are working together to ensure a better future for our planet with environmentally friendly programs and products. Our internal effort includes a recycling program in which we recycled over 22,000 Kg of paper, 17,000 Kg of cardboard and over 454 Kg of batteries.

We are working toward finding ways to conserve energy, building our offering of "green" products and providing recycling options for our customers. Look for the leaf design signifying our "green" offerings.

ENZYMAX EARTH Detergent Enjoy all of the same great benefits of Enzymax detergents with Enzymax Earth detergents, as an increased focus on the environment.

SPORECLEAR is non-alcohol based, free of aldehydes, chlorine, phenols, peroxide and nonylphenol ethoxylates, (NPEs), making them compatible with all washable surfaces.

LEAD & LATEX FREE AUTOCLAVE TAPE Making it safe for the environment, your staff and your patients and does not require special hazardous disposal.

ENZYMAX DUAL ENZYME ULTRASONIC AND PRESOAK DETERGENT

Enzymax Dual Enzyme Ultrasonic Detergent is specifically formulated for efficient ultrasonic cleaning or pre-soaking of cassettes and instruments. Contains Steelgard agents to protect instruments from harmful minerals naturally found in water that contribute to buildup, spotting and corrosion. **Can also be used as an evacuation system cleaner.**

Can also be used as a linen spot remover. Easy to use – simply spray directly onto instruments or fabric, allow to soak, rinse, then process as usual.



ENZYMAX SPRAY GEL

24 fl. oz/709 ml IMS-1229
450 Sprays

Dual enzyme gel with corrosion inhibitor acts as a ready-to-use instrument pre-cleaner. Maintains moisture to help prevent debris from drying on instruments before cleaning and helps dissolve blood, tissue, and other debris. Can also be used as a linen spot remover.

ENZYMAX LIQUID

A fast-acting, concentrated, liquid formula. Available in easy to pump quart, gallon and liquid unit dose configurations.

Box of 40 Liquid Packets IMS-1222
Box of 40 Packets of 10 ml (makes ca. 140 liters)

Quart Bottle IMS-1224

Quart bottle, 950 ml, with pump (makes ca. 120 liters)

Quart Pump IMS-1224P

Pump for 950 ml Bottle

Gallon bottle, 3,8 l IMS-1226
(makes ca. 480 litres)

Gallon Pump, 3,8 l IMS-1226P

Pump for 3,8 l Bottle

ENZYMAX POWDER

1.76LB/800g Powder IMS-1230
Makes 50 Gallons/189 liters

6 containers of 1.76LB/800g Powder IMS-1230C
Makes 300 Gallons/1,134 liters

ENZYMAX PAX SINGLE USE PACKETS

32 Dissolvable Single-Use Powder Packets IMS-1232
Makes 32 Gallons/121 liters

96 Dissolvable Single-Use Powder Packets IMS-1233
Makes 96 Gallons/363 liters

INSTRUCTION FOR USE: ENZYMAX

ENZYMAX Dual Enzyme Ultrasonic Detergent is specifically formulated for efficient ultrasonic cleaning or pre-soaking of cassettes and instruments. Contains Steelgard agents to protect instruments from harmful minerals naturally found in water that contribute to buildup, spotting and corrosion. Can also be used as an evacuation system cleaner.

ENZYMAX is perfectly suitable for the removal of blood, proteins and other substances. Especially for surgical, endodontic and all other instruments that get contaminated with blood.

Do not use disinfectants together with ENZYMAX.

Safety Data Sheets: www.hu-friedy.eu

EASY APPLICATION:

Your practice and patients deserve the best, so count on Enzymax for a superior level of instrument cleaning.

Wear gloves and eye protection. Use with water 32°-60°C, demineralized water is ideal. Rinse, dry thoroughly instruments and cassettes, then proceed with sterilization. Change solution at least once a day.

Instrument presoak:

Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Soak for 3-5 minutes. Rinse, dry thoroughly and proceed with sterilization.

Ultrasonic bath:

Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. IMS-1222: 1 pck. (10 ml): 1 packet is sufficient for 3,5 liters of water.

Rinse, dry thoroughly instruments and cassettes, then proceed with sterilization.

Linen presoak:

Approx. 9 caps for 4 liters or 2 packets or 2 liquid packs or pump 9 times. Soak clothing for 10 minutes. Proceed with normal washing.

Enzymax Spray Gel:

1. Spray directly onto instrumentation after procedure or prior to washing process.
2. Evenly coat instruments
3. Allow instruments to soak
4. Rinse before processing
5. Follow normal cleaning procedures

Can also be used as a linen spot remover.

Easy to use—simply spray directly onto instruments or fabric, allow to soak, rinse, then process as usual.

For more information about our products visit us hu-friedy.eu

Hu-Friedy Mfg. Co., LLC | 3232 N. Rockwell Street | Chicago, IL 60618 | USA
Customer Care Department: Free Call: 00800 48 37 43 39
Free Fax: 00800 48 37 43 40 | Web site: Hu-Friedy.eu

© 2015 Hu-Friedy Mfg. Co., LLC. All rights reserved. HF-3296B/0915

